# Respirelles

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## COMPOSITION

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Nasal Dilator

## COMPOSITION

The **Respirelles** medical device is made of bisphenol-free, phthalate-free TM9MED non-allergic surgical PVC.

## PACKAGING

Packaged in a circular PVC box containing 5 pairs of **Respirelles** (nasal dilators) combined in pairs. The diameters are indicated by references I to IIIII on the inside of the device. These various diameters enable the user to select the one which fits the 2 nostrils as the nostrils of nose can have differing diameters (orifices).

## DIRECTIONS FOR USE

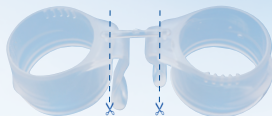
**Respirelles** is a simple, natural method for improving breathing and benefiting from all the advantages of proper oxygenation enabling improvements in:

- night-time breathing • quality of sleep • sleep after a meal • concentration • vocal fatigue on awakening • exercise performance.

As well as decreased: laryngopharyngeal reflux • dryness of the mouth on awakening • morning fatigue • snoring (suppression in some cases). It enables the ENT specialist to better pinpoint the various origins of the snoring and decide on a surgical procedure on the uvula and/or nasal pathology, if this becomes actually necessary after the "**Respirelles**" test is performed.

## INSTRUCTIONS

**Respirelles** are combined in pairs and must be separated before use. Choose the diameter appropriate for each nostril (if different). This should dilate the nostrils (check by looking in a mirror, as the diameter selected enables more effective use of the device). Before using **Respirelles**, it is recommended that the nose be cleaned with a saline nasal solution. Should discomfort be experienced while initially wearing the devices, you can opt to wear only one for several days to build tolerance. For optimum discretion you can cut the tabs at the desired point (as in the photo below).

## CARE AND USAGE PERIOD

After use, clean the **Respirelles** dilators under running water. They can be sterilised in boiling water or in blue light if used in a hospital environment. If the **Respirelles** dilators become misshapen with use, it is recommended that they be changed for maximum effectiveness. With daily use, **Respirelles** dilators last for 1 month.

## PRECAUTIONS DURING USE

Not recommended for use with nasal spurs, infection or nasal drip from colds.

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